

StatStrip Glucose

External Quality Control Program

Directions for performing test:

- Select the appropriate sample as indicated on the pipette (blue solution) and cut off the tip.
- Run the control solution as a patient test, scanning the sample barcode as the patient ID.
- Discard the first few drops and apply a large drop to the test strip ensuring there are no air bubbles.
- Test all of your StatStrip meters using the same solution.
- Once analysis is complete, fill out this form and fax to iCCnet on (08) 7117 0635 or email Health.iCCnetSupport@sa.gov.au.

	Device Serial Number	EQA:	EQA:
Device 1			
Device 2			
Device 3			
Device 4			

Date Analysed:

Operator:

Site Name:

Please fax results to iCCnet SA on 08 7117 0635